



SEM

SEM TRAINERS & SYSTEMS

WHERE TIME MOVES AHEAD TO KEEP PACE WITH KNOWLEDGE

SEM- Scientific Educative Methods in Science, Engineering & Medicine

Mobile : +91 88495 63724

Mobile 1: +91 98791 03905

Email: sem@semtrainers.com

Website: www.semtrainers.com



Catheterization Simulator PRO, male

Item No. 1020230 [P93P-M]

Weight 3.5kg

Dimensions 50 x 42 x 30 cm

Brand 3B Scientific

[Read More](#)

SKU:

Categories: Catheterization



Product Description

With the male P93 PRO catheterization trainer, male bladder catheterization procedures can be realistically demonstrated, practiced and assessed. The genital inserts are placed into the anatomically correct lower abdominal model and held in place with magnets. The material is soft and flexible: the foreskin is movable and the penis can be stretched. This way the student can practice all the necessary

gestures for catheterization (e.g. disinfection). The material gives a realistic feeling when inserting and removing the catheter into the bladder with realistic resistance and insertion depths. **Direct learning:** When the catheter has been correctly inserted, fluid runs out, just like with a real patient. The catheter can also be checked through the transparent bladder. The abdominal wall can be removed so that the process can be viewed better. **Various levels of difficulty:** On the male insert, three different levels of narrowing of urethra can be set. In the narrowest setting, catheterization is not possible, leading to the conclusion that a suprapubic catheter is needed. On the PRO version, a suprapubic catheter has already been put in place, and the student can practice cleaning it and taking care of it with the trainer. **All the benefits of the male P93 PRO catheterization simulator at a glance:**

- Male catheterization with realistic resistance
- 3 levels of adjustable narrowing of urethra
- Soft and movable foreskin (for practicing disinfection, for example)
- Anatomically realistic pelvic structure
- Liquid outflow if catheterization is successfully carried out
- The transparent bladder can be checked, and the abdominal wall can be removed
- Includes suprapubic catheter for practicing, cleaning and caring for it
- Stands securely on the table thanks to the non-slip feet
- Magnetic connectors for quick set-up and dismantling
- Easy to clean and maintain, and can be completely taken apart
- A sponge on the inside prevents a build-up of moisture

Delivery contents:

- Realistic lower abdominal model
- Male catheterization set with transparent bladder
- Bottle and tube connectors
- Carry case

Note: For catheterization, we recommend using catheters with only one or two drainage eyes opposite each other. Both silicone and PVC catheters can be used. We recommend the following catheters:

- Urethral narrowing “open” = 16 CH catheter or 14 CH balloon catheter
- Urethral narrowing “partial narrowing” = 14 CH catheter or 12 CH balloon catheter

Catheters with a curved tip should not be used! Tip: According to the guidelines for correct catheterization, even during the simulation, lubricating gel (not included) should be placed into the urethra to ensure that the catheter can be inserted smoothly. Alternatively, some lubricating gel can be placed on the tip of the catheter.

Why Choose the Catheterization Simulator PRO, Male, for Your Skills Lab in India?

The Catheterization Simulator PRO, male (Item No. 1020230 [P93P-M], 3.5 kg, 50 × 42 × 30 cm, Brand: 3B Scientific) is 3B Scientific's most complete male catheterization trainer, combining an anatomically realistic lower abdominal model with the full teaching progression from initial insertion through ongoing suprapubic catheter care. Genital inserts mount magnetically, with a soft, movable foreskin and stretchable penis for practicing the complete procedural sequence, including disinfection technique.

Like the male BASIC trainer, PRO offers **three adjustable levels of urethral narrowing**, with the narrowest setting deliberately non-catheterizable — teaching students to recognize when urethral catheterization has failed and a suprapubic approach is the correct clinical decision. Unlike BASIC, PRO takes this a step further: **a suprapubic catheter comes pre-installed**, so students can practice its ongoing care and cleaning, completing the full clinical pathway from initial attempt through alternative-access management.

The removable abdominal wall gives instructors a clearer view of catheter placement during demonstration, and successful urethral catheterization produces genuine fluid outflow, visually confirmable through the transparent bladder.

Features:

- Male catheterization with realistic resistance
- 3 adjustable levels of urethral narrowing, including a non-catheterizable level for suprapubic-catheter decision training
- Soft, movable foreskin
- Anatomically realistic pelvic structure with removable abdominal wall
- Fluid outflow confirms successful catheterization
- **Pre-installed suprapubic catheter** for practicing care and cleaning
- Non-slip feet for secure table placement
- Magnetic connectors for quick setup/teardown
- Fully disassembles for cleaning; internal sponge prevents moisture buildup

Delivery contents: Realistic lower abdominal model, male catheterization set with transparent bladder, bottle and tube connectors, carry case.

Recommended catheters (not included), for the "open" and "partial narrowing" levels:

- Urethral narrowing "open": 16 CH catheter or 14 CH balloon catheter
- Urethral narrowing "partial narrowing": 14 CH catheter or 12 CH balloon catheter

- Use catheters with only one or two opposing drainage eyes. Both silicone and PVC catheters are suitable. **Do not use catheters with a curved tip.**
- The narrowest setting is intentionally non-catheterizable — this level teaches recognition of when a suprapubic approach is needed.
- Lubricating gel is recommended (not included) for smooth insertion.

Why Buy from SEM Trainers? SEM Trainers & Systems is an authorized distributor of 3B Scientific products in India with over 30 years of experience supplying medical simulators and clinical skills training equipment to 1,500+ institutions across 28 states. We provide pan-India delivery and dedicated after-sales support.

Q: How does this compare to the male BASIC trainer? A: PRO adds a full anatomical abdominal model with removable wall, a carry case, and — most significantly — a pre-installed suprapubic catheter for care and cleaning practice, extending training beyond BASIC's insertion-only scope.

Q: What's the purpose of the three narrowing levels? A: They train procedural technique and clinical judgment together — the narrowest level is intentionally non-catheterizable, and PRO lets students then practice the resulting suprapubic catheter care.

Q: How do I order from SEM Trainers? A: Contact us at sem@semtrainers.com or call +91-88495 63724 for pricing and availability.