

INSTRUCTION MANUAL

**Life/form® Basic Child  
CRiSis™ Trainer Torso with  
Advanced Airway  
Management**

LF03633



*Nasco*  
HEALTHCARE

# About the Simulator

The Nasco Healthcare Life/form® Basic Child CRiSis™ Trainer Torso with Advanced Airway Management is the most realistic simulator available for the training of intubation and other airway management skills.

Nasco Healthcare has taken great care to create an airway management trainer that is anatomically correct in respect to both size and detail. Landmarks include nostrils, teeth, tongue, oral and nasal pharynx, larynx, epiglottis, arytenoids, false and true vocal cords, trachea, esophagus, lungs, and stomach.

The Nasco Healthcare Life/form® Basic Child CRiSis™ Trainer Torso with Advanced Airway Management allows you to practice oral, digital, and nasal intubation, as well as suction techniques. The simulator was designed to use an uncuffed endotracheal tube measuring up to 5.0 mm I.D. With proper care, your Nasco Healthcare Life/form® Basic Child CRiSis™ Trainer Torso with Advanced Airway Management will provide years of valuable service.

# List of Components

• Nasco Healthcare Life/form® Basic Child CRiSis™ Trainer Torso with Advanced Airway Management

## SET UP

Your Life/form® Child CRiSis™ Airway Management Trainer is ready to use upon delivery. Simply remove from carton and unwrap packaging material.

## LUBRICATION

Before intubating the Life/form® Basic Child CRiSis™ Trainer Torso with Advanced Airway Management, lubricate the simulator with the Nasco lubricant provided.

(See Figure 1)



Figure 1

## CLEANING THE FACE

Normal soil can be removed from the face of the trainer with mild soapy water. Nasco Cleaner (LF09919) will remove stubborn stains. Simply apply Nasco Cleaner to soiled area and wipe clean with a soft cloth or paper towel.

**Note:** Do not use Nasco Cleaner around the mouth and nostrils, as the residue of cleaner could be toxic.

Actual product may vary slightly from photo. Nasco healthcare reserves the right to change product color, materials, supplies, or function as needed.

## CAUTIONS

Nasco Healthcare recommends the use of the enclosed aerosol lubricant or a similar vegetable-based lubricant for use with our Nasco Healthcare Life/form® Basic Child CRiSis™ Trainer Torso with Advanced Airway Management.


**DO NOT** use a silicone or similar lubricant. This will cause your simulator to dry and crack, and will automatically void Nasco Healthcare's warranty on the trainer. Never place the trainer on any kind of printed paper or plastic. These materials will transfer indelible stains. Ballpoint pens will also make indelible stains.

## AVAILABLE SUPPLIES

- LF03644** Pump Spray Lubricant
- LF03627** Replacement Lung Set, Child
- LF03628** Replacement Stomach, Child
- LF09919** Nasco Cleaner

## ADD-ON COMPONENTS

- LF03611** Defibrillation Chest Skin
- LF03612** Life/form® Child CRiSis™ Manikin Injectable Training Arm
- LF03613** Blood Pressure Arm
- LF03614** Intraosseous Infusion/Femoral Access
- LF03634** Plain Left Leg

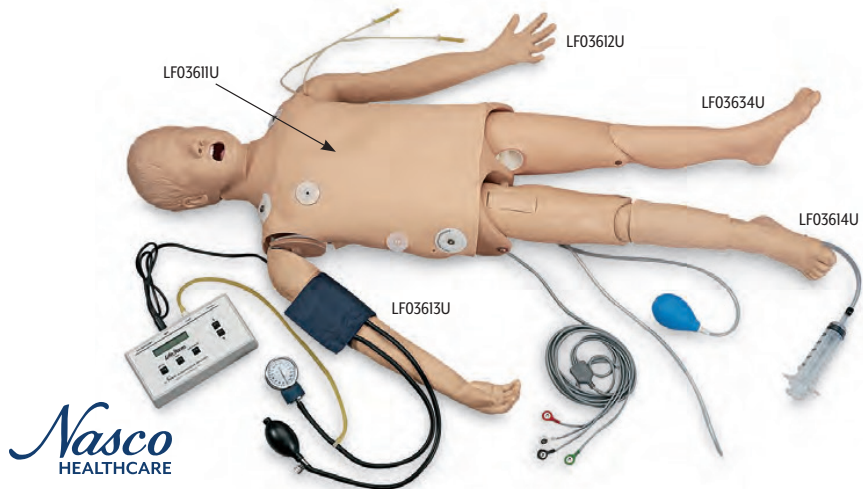

**NASCO HEALTHCARE**  
16 Simulaids Drive  
Saugerties, NY 12477  
1-833-NASCOHC (627-2642)  
info@nascohealthcare.com  
www.nascohealthcare.com