


INSTRUCTION MANUAL

Testicular Exam Simulator

LF01143

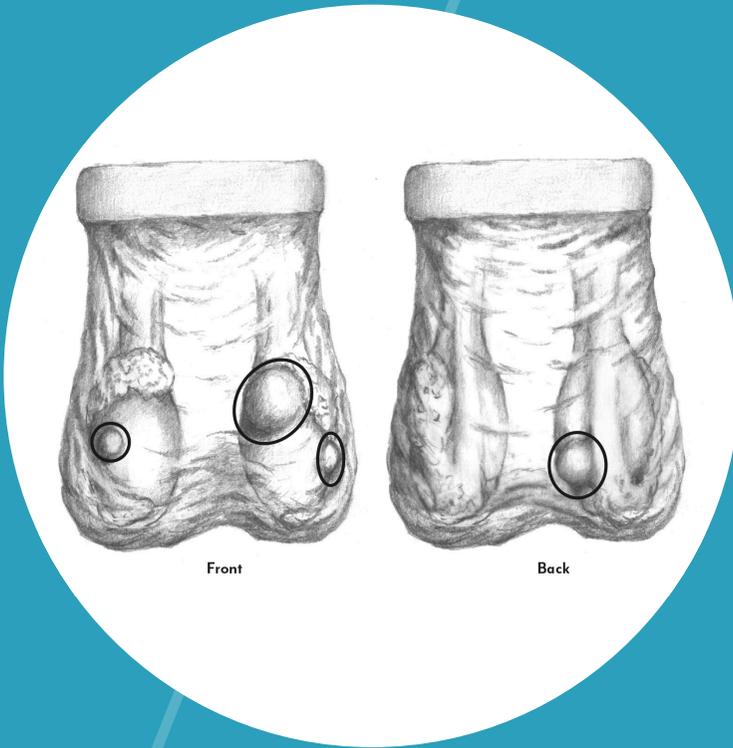

5 Year Warranty

Nasco
HEALTHCARE

About the Simulator

For greatest realism, the outer skin of the simulator is soft and thin, and the underlying structures are delicate. Handle the model gently. You may find that a light dusting of baby powder gives the surface a more realistic feel. Be especially careful not to puncture the skin in any way, as this will result in the loss of the internal lubricant.

The simulator may be washed in a mild detergent solution if it becomes soiled. Rinse well and pat dry. Cleaner is recommended to remove more serious stains. Never set the simulator on any printed material, as this will cause permanent stains on the skin. The soft plastic may also stain paper or damage some finished surfaces after prolonged contact. For longest life and best results, clean, wrap, and store the simulator properly after each use.

SUPPLIES/ REPLACEMENT PARTS

LF09919 Cleaner

LF01143(A) Testicles Only



Nasco Healthcare
16 Simulaid Drive
Saugerties, NY 12477
1-833-NASCOHC (627-2642)
info@nascohealthcare.com
www.nascohealthcare.com